

THIS COMPONENT PRINTED TWO SIDES

SPECIAL INSTRUCTIONS		BAUSCH & LOMB VISION CARE PACKAGING/GRAPHIC ENG. 1400 NORTH GOODMAN STREET ROCHESTER, N. Y. 14603-0450 TEL: 716-338-5474 FAX: 716-338-8959	APPROVALS / DATE
DIE LINE DOES NOT PRINT	COATING PER SPECIFICATION	PKG/GRPH _____	_____
PRINTED BAR CODES MUST BE VERIFIED READABLE BY PRINTING VENDOR		MKT/GS _____	_____
		REG AFF _____	_____
		QA _____	_____
		DCO NUMBER _____	_____
		DATE _____	_____
THE INFORMATION LISTED BELOW MUST APPEAR ON ALL VENDOR PROOFS (COLORKEYS, MATCHPRINTS/CHROMALINS) OR THEY WILL BE REJECTED			
PLACED PICTURES			
AW #LB9135/06	DESCRIPTION Boston Lens II & IV Patient Inst./Pkg. Insert	SPEC. NO. 4549	
NUMBER OF PRINTING PLATES 1	TOTAL NUMBER OF PMS COLORS 1	TOTAL AW PAGES 2	
PMS COLORS		PROCESS MATCH COLORS	
1. PMS 326 AQUA			

PATIENT INSTRUCTIONS

Boston® IV
(itafocon B)

Boston® II
(itafocon A)

**Silicone Acrylate
Rigid Gas Permeable
Contact Lenses
For
Daily Wear**

**Bausch & Lomb
Boston®
Materials**

CAUTION:

Federal (USA) Law restricts this device to sale by or on the order of a licensed practitioner.

TABLE OF CONTENTS

Introduction
Wearing Restrictions and Indications
Contraindications
Warnings
Precautions
Adverse Effects
Personal Cleanliness and Lens Handling
 Handling the Lenses
 Preparing the Lens for Wearing
 Placing the Lens on the Eye
 Centering the Lens
 Removing the Lens
Caring for Your Lenses
 Cleaning, Disinfecting, Storage and Rewetting/Lubricating
 Basic Instructions
 Care for a Sticking (Nonmoving) Lens
 Lens Case Cleaning and Maintenance
 Emergencies
 Instructions for Monovision Wearer
 Prescribed Wearing Schedule
 Appointment Schedule

INTRODUCTION

Your eyecare practitioner has fit you with contact lenses known as Boston® IV or Boston® II. These lenses are manufactured from Boston IV and Boston II lens material. It is essential that you follow the recommended handling, cleaning and storage procedures. Failure to do so may eventually impair the performance of your lenses.

Boston® IV (itafocon B) and Boston® II (itafocon A) are Rigid Gas Permeable Contact Lens materials composed of methyl methacrylate dimethylsiloxane copolymer. Boston IV lens materials are available in clear and blue. The blue Boston IV lens materials contain the color additive D & C Green No. 6. Boston II lens materials are available in clear, blue and green. The blue Boston II lens material contain the color additive D & C Green No. 6, the green Boston II lens material contain the color additive D & C Green No. 6 and C.I. Solvent Yellow No. 18.

WEARING RESTRICTIONS AND INDICATIONS

Boston IV and Boston II Contact Lenses are indicated for daily wear for the correction of refractive ametropia (myopia, hyperopia, astigmatism, presbyopia and keratoconus) in aphakic and not aphakic persons with nondiseased eyes. The lens may be disinfected using a chemical disinfection (not heat) system only.

Boston IV and Boston II contact lenses described in this booklet should be removed from your eyes for routine cleaning and disinfecting as prescribed by your eyecare practitioner. DO NOT WEAR YOUR BOSTON IV or BOSTON II CONTACT LENSES WHILE SLEEPING.

CONTRAINDICATIONS (REASONS NOT TO USE)
DO NOT USE BOSTON IV or BOSTON II Contact Lenses when any of the following conditions exist:

- Acute or subacute inflammation or infection of the anterior chamber of the eye
- Any eye disease, injury, or abnormality that affects the cornea, conjunctiva, or eyelids
- Severe insufficiency of lacrimal secretion (dry eyes)
- Corneal hyposthesia (reduced corneal sensitivity), if not aphakic
- Any systemic disease that may affect the eye or be exaggerated by wearing contact lenses
- Allergic reactions of ocular surfaces or adnexa that may be induced or exaggerated by wearing contact lenses or using contact lens solutions
- Allergy to any ingredient in a solution which is to be used to care for Boston IV or Boston II contact lenses
- Any active corneal infection (bacterial, fungal, or viral)
- Red or irritated eyes

WARNINGS

- Problems with contact lenses and lens care products could result in **serious injury** to the eye. It is essential to follow your eyecare practitioner's directions and all labeling instructions for proper use of lenses and lens care products, including the lens case. Eye problems, including corneal ulcers, can develop rapidly and lead to **loss of vision**.

CAUTION: Do not wear lenses are not indicated for overnight wear, and should not be worn while sleeping. Clinical studies have shown that the risk of serious adverse reactions is increased when these lenses are worn overnight.

- Studies have shown that contact lens wearers who are smokers have a higher incidence of adverse reactions than nonsmokers.
- If you experience eye discomfort, excessive tearing, vision changes, or redness of the eye, **immediately remove lenses** and promptly contact your eyecare practitioner.

PRECAUTIONS

You should carefully adhere to the following care regimen and safety precautions:

- Before leaving the eyecare practitioner's office, you should be able to properly remove lenses or should have someone else available who can remove the lenses for you.

DO NOT USE BOSTON IV or BOSTON II CONTACT LENSES WHEN ANY OF THE FOLLOWING CONDITIONS EXIST:

- You should remove your lenses immediately if your eyes become red or irritated.
- Different solutions often cannot be used together, and not all solutions are safe for use with all lenses. Use only recommended solutions.
- Do not heat the wetting/soaking solution and lenses. Keep them away from extreme heat.
- Always use **fresh unexpired** lens care solutions.
- Always follow directions in the package insert for the use of contact lens solutions.
- Use only a chemical (not heat) lens care system. Use of a heat (thermal) care system can warp the Boston IV and Boston II contact lenses.
- Sterile unexpired solutions, when used, should be discarded after the time specified in the labeling directions.
- Do not use saliva or anything other than the recommended solutions for lubricating or wetting lenses.
- Always keep the lenses completely immersed in the recommended storage solution when the lenses are not being worn (inserted). If dry storage is desired to store the lenses for a longer period of time, they must first be cleaned, rinsed with water and carefully dried by blotting with a soft lint free tissue prior to being placed in a clean, dry lens storage case. Ideally, these lenses should be cleaned and disinfected overnight prior to insertion.
- If the lens sticks (stays moving) on the eye, follow the recommended directions on Care for a Sticking Lens. The lens should move freely on the eye for the continued health of the eye. If non-movement of the lens continues, immediately consult your eyecare practitioner.
- Always wash and rinse hands before handling lenses. Do not get cosmetics, lotions, soaps, creams, deodorants, or sprays in the eyes or on the lenses. It is best to put on lenses before putting on makeup. Water-based cosmetics are less likely to damage lenses than oil-based products.
- Do not touch contact lenses with the fingers or hands if the hands are not free of foreign material, as microscopic scratches on the lenses may occur, causing distorted vision and/or injury to the eye.
- Carefully follow the handling, insertion, removal, cleaning, disinfecting, storing and wearing instructions that follow for Boston IV and Boston II Contact Lenses and those provided by your eyecare practitioner.
- Never wear lenses beyond the period recommended by your eyecare practitioner.
- If aerosol products such as hair spray are used while wearing lenses, exercise caution and keep eyes closed until the spray has settled.
- Always handle lenses gently and avoid dropping them on hard surfaces.
- Avoid all harmful or irritating vapors and fumes while wearing lenses.
- Ask your eyecare practitioner about wearing lenses during water activities and other sports.
- Inform your health care practitioner (doctor) that you wear contact lenses.

NEVER use tweezers or other tools to remove lenses from the lens case unless specifically indicated for that use. To remove the lens from the case, pour the solution containing the lens into the palm of your hand.

- Do not touch the lens with fingernails.
- Always contact your eyecare practitioner before using any medicine in the eyes.
- Always inform your employer that you wear contact lenses. Some jobs may require use of eye protection equipment or may require that you not wear contact lenses.
- As with any contact lens, follow-up visits are necessary to assure the continuing health of your eyes. Follow your eyecare practitioner's instructions to a recommended follow-up schedule.

ADVERSE EFFECTS

The following problems may occur:

- Eyes stinging, burning, itching (irritation) or other eye pain
- Comfort is less than when lens was first placed on the eye
- Feeling of something in the eye such as a foreign body, scratched area
- Excessive watering (tearing) of the eyes
- Unusual eye secretions
- Redness of the eyes
- Reduced sharpness of vision (poor visual acuity)
- Blurred vision, rainbows, or halos around objects
- Sensitivity to light (photophobia)
- Dry eyes

Immediately remove lenses.

- If the discomfort or problem stops, then look closely at the lens. If the lens is in any way damaged, do not put the lens back on the eye. Place the lens in the storage case and contact your eyecare practitioner. If the lens has dirt, an eyelash, or other foreign body on it, or the problem stops and the lens appears undamaged, you should thoroughly clean, rinse, and disinfect the lenses, then reinsert them. After reinsertion, if the problem continues, **immediately remove the lenses and consult your eyecare practitioner.**

PERSONAL CLEANLINESS FOR LENS HANDLING

When any of the above problems occur, a serious condition such as infection, corneal ulcer, neovascularization, or keratitis may be present. You should **keep the lens off the eye and seek immediate attention.**

PERSONAL CLEANLINESS FOR LENS HANDLING

1. Preparing the Lens for Wearing

It is essential that you learn and use good hygienic methods in the care and handling of your new lenses. Cleanliness is the first and most important aspect of proper contact lens care. In particular, your hands should be clean and free of any foreign substances when you handle your lenses. The procedures are:

- Always wash your hands thoroughly with a mild soap, rinse completely, and dry with a lint-free towel before touching your lenses.

4. **Centering the Lens**

Very rarely, a lens that is on the cornea will be displaced onto the white part of the eye during lens wear. This can also occur during placement and removal of the lenses if the correct techniques are not performed properly. To center a lens follow one of the procedures below:

- Close your eyelids and gently massage the lens into place through the closed lids.
- OR
- Gently push the off-centered lens onto the cornea while the eye is open using finger pressure on the upper or lower lid next to the edge of the lens.

5. Removing the Lens

Before removing your lenses, it is recommended that you have the following items available:

1. A lens storage case.
2. **Two Bottle Care System**
Boston ADVANCE® Cleaner or Boston® Cleaner AND Boston ADVANCE® Comfort Formula Conditioning Solution or Boston® Conditioning Solution.

OR

One Bottle Care System
Boston SIMPLIS® Multi-Action Solution (Contains Protein, Cleans, Disinfects, Conditions & Cushions)
A clean towel.

Two-Finger Method

1. Place a towel under your eye to catch the lens.
2. Place the tip of the forefinger of one hand on the middle of the upper lid margin and the forefinger of the other hand on the middle of the lower lid margin.
3. Press the lid margin inward and then together. The lens should be removed out of your eye onto your hand or towel.
4. The lens may come out but be wedged on your eyelid or hand or be decentered onto the white part of your eye. If the latter occurs, recenter the lens onto your cornea before repeating the removal procedure.

Blink Method

Seat yourself at a table covered with a clean towel and lean over until you are looking down at the surface.

1. Place your index finger at the outer junction of your upper and lower lids, stretch the skin outward and slightly upward. Do not allow your lid to slide over the lens!
2. Blink briskly. The lens will be pinched by the pressure of your eyelids and the lens will pop out into the clean surface of the towel, or you may catch the lens in the palm of your hand.
3. Remove the other lens by following the same procedure.

d. Follow the required lens care procedures described under the heading, **CARING FOR YOUR LENSES (CLEANING, RINSING, DISINFECTING, STORAGE AND REWETTING/LUBRICATING).**

Note: If these methods for removing your lenses are difficult for you, your eyecare practitioner will provide you with an alternate method.

CARING FOR YOUR LENSES (CLEANING, RINSING, DISINFECTING, STORAGE AND REWETTING/LUBRICATING)

1. Basic Instructions

For continued safe and comfortable wearing of your lenses, it is important that you first **clean and rinse**, then **disinfect** your lenses after each removal, using the care regimen recommended by your eyecare practitioner. **Cleaning and rinsing** are necessary to remove mucus, secretions, films, or deposits which may have accumulated during wear. The ideal time to clean your lenses is immediately after removing them. **Disinfecting** is necessary to destroy harmful germs.

2. Lens Case Cleaning and Maintenance

Contact lens cases can be a source of bacterial growth. Lens cases should be emptied, cleaned, rinsed with solutions recommended by the lens case manufacturer or the eyecare practitioner, and allowed to air dry after each use. Lens cases should be replaced at regular intervals as recommended by the lens case manufacturer or the eyecare practitioner.

3. Emergencies

If chemicals of any kind (household products, gardening solutions, laboratory chemicals, etc.) are splashed into your eyes, you should **RUSH YOUR EYES IMMEDIATELY WITH TAP WATER. THEN REMOVE YOUR LENSES PROMPTLY IF POSSIBLE AND IMMEDIATELY CONTACT YOUR EYECARE PRACTITIONER OR VISIT A HOSPITAL EMERGENCY ROOM WITHOUT DELAY.**

INSTRUCTIONS FOR THE MONOVISION WEARER

You should be aware that, as with any type of lens correction, there are advantages and compromises to monovision contact lens therapy. The benefits of clear near vision in all situations that is available with monovision may be accompanied by a vision compromise that may reduce your visual acuity and depth perception for distances and near tasks. Some patients have experienced difficulty adapting to monovision. Symptoms, such as mild blurred vision, dizziness, headaches and a feeling of slight imbalance, may last for a brief minute or for several weeks as adaptation takes place. The longer these symptoms persist, the more serious your adaptation difficulties may be. You should avoid visually demanding situations during the initial adaptation period. It is recommended that you first wear these contact lenses in familiar situations which are not visually demanding. For example, be a passenger rather than a driver of an automobile during the first few days of lens wear. It is recommended that you only drive with monovision correction if you pass your state drivers' license requirements with monovision correction.

WARNING: Boston IV and Boston II Contact Lenses are **NOT** intended for overnight extended wear.

The lens care products listed below are recommended by Bausch & Lomb for use with either your Boston IV or Boston II Contact Lenses. Your eyecare practitioner may recommend alternate products that are appropriate for you to use with your Boston IV or Boston II Contact Lenses.

LENS CARE TABLE

Product Purpose	Lens Care System (Chemical / Heat)
Clean	Boston ADVANCE® Cleaner or Boston® Cleaner
Disinfect	Boston ADVANCE® Comfort Formula Conditioning Solution or Boston® Conditioning Solution
Store	Boston ADVANCE® Comfort Formula Conditioning Solution or Boston® Conditioning Solution
Multi-Action (Clean, Condition, Disinfect, Rinse and Cushion)	Boston SIMPLIS® Multi-Action Solution
Lubricate/Rewet	Boston® Rewetting Drops
Weekly Enzymatic Clean	Boston® ONE STEP Liquid Enzymatic Cleaner

Note: Some solutions may have more than one function, which will be indicated on the label. Read the label on the solution bottle, and follow instructions.

- Clean one lens first always the same lens first to avoid mix-up! and rinse the lens thoroughly as recommended by your eyecare practitioner to remove the cleaning solution, mucus, and film from the lens surface. Follow the instructions provided in the cleaning solution labeling. Put that lens into the correct chamber of the lens storage case. Then repeat the procedure for the second lens.
- After cleaning, disinfect lenses using the above recommended system by your eyecare practitioner and/or the manufacturer. Follow the instructions provided in the disinfection solution labeling.
- To store lenses, disinfect and leave them in a recommended disinfecting/conditioning solution when the lenses are not being worn. If you discontinue wearing your lenses, but plan to begin wearing them again after a few weeks, ask your eyecare practitioner for a recommendation on how to store your lenses.
- Boston IV and Boston II Contact Lenses cannot be heat thermally disinfect.
- After removing your lenses from the lens case, empty and rinse the lens storage case. Some monovision lens wearers will never be fully comfortable functioning under low levels of illumination, such as driving at night. If this happens, discuss with your eyecare practitioner whether you should have additional contact lenses prescribed so that both eyes are corrected for distance when sharp distance vision is required.
- If you require very sharp near vision during prolonged close work, you may want to discuss with your eyecare practitioner having additional contact lenses prescribed so that both eyes are corrected for near when sharp near vision is required.
- Some monovision lens wearers require supplemental spectacles to wear over the monovision correction to provide the clearest vision for critical tasks. You should discuss this with your eyecare practitioner.
- It is important that you follow your eyecare practitioner's suggestions for adaptation to monovision contact lens therapy. You should discuss any concerns that you may have during and after the adaptation period.
- **The decision to be fit with a monovision correction is most appropriately left to the eyecare practitioner in conjunction with the patient after carefully considering and discussing your needs.**

CONSIDERATIONS FOR BIFOCAL LENSES

Patients who are considering bifocal contact lenses should be highly motivated and must be informed of the benefits as well as the problems that may be encountered while adapting to bifocal contact lens wear.

Your eyecare practitioner may discuss the following with you:

1. **Adaptation**
Both bifocal spectacle and bifocal contact lens wearers need to learn to adapt to proper head positioning. The bifocal patient must position the head upright while crossing the eyes downward to read. Once the bifocal patient has adapted, proper positioning becomes effortless.
2. **Driving at Night**
Bifocal contact lens wearers should experience night vision before actually driving while wearing their lenses.
3. **Fare at Night**
Bifocal contact lenses wearers may experience fare at night. This may occur with certain lens designs. With time, bifocal contact lens wearers adapt to this situation.
4. **Visual Expectation**
Bifocal contact lens wearers may experience visual acuities less than could be achieved with bifocal spectacles.

EXAMPLE OF WEARING AND APPOINTMENT SCHEDULES

DAY	WEARING TIME (HOURS)*
1	4 to 8 Hours
2	6 to 10 Hours
3	8 to 14 Hours
4	10 to 15 Hours
5	12 to All Waking Hours
6 and after	All Waking Hours

* If the lenses continue to be well-tolerated.

APPOINTMENT SCHEDULE

Minimum number of hours lenses to be worn at time of appointment.

Month	Year	Time	Date

PATIENT/EYECARE PRACTITIONER INFORMATION

Eyecare Practitioner Information

Practitioner Name: _____

Practice Name: _____

Practitioner Address: _____

Practitioner Phone Number: _____

Recommended Lens Care Regimen: _____

Cleaning Solution: _____

Conditioning Solution: _____

Rewetting Solution: _____

Weekly Enzymatic Cleaner: _____

IMPORTANT: In the event that you experience any difficulty wearing your lenses or you do not understand the instructions given you, DO NOT WAIT for your next appointment. TELEPHONE YOUR EYECARE PRACTITIONER IMMEDIATELY.

© Bausch & Lomb Incorporated
1400 North Goodman Street
Rochester, NY 14609
1-800-355-4730

* denotes registered trademarks of Bausch & Lomb Incorporated.

Print Date: 09/07

LB9135/06

THIS COMPONENT PRINTED TWO SIDES

SPECIAL INSTRUCTIONS		BAUSCH & LOMB VISION CARE PACKAGING/GRAPHIC ENG. 1400 NORTH GOODMAN STREET ROCHESTER, N. Y. 14603-0450 TEL: 716-338-5474 FAX: 716-338-8959	APPROVALS / DATE	
DIE LINE DOES NOT PRINT	COATING PER SPECIFICATION		PKG/GRPH	_____
PRINTED BAR CODES MUST BE VERIFIED READABLE BY PRINTING VENDOR			MKT/GS	_____
			REG AFF	_____
			QA	_____
		DCO NUMBER	_____	
		DATE	_____	

THE INFORMATION LISTED BELOW MUST APPEAR ON ALL VENDOR PROOFS (COLORKEYS, MATCHPRINTS/CHROMALINS) OR THEY WILL BE REJECTED

PLACED PICTURES			
AW #LB9135/06	DESCRIPTION Boston Lens II & IV Patient Inst./Pkg. Insert	SPEC. NO. 4549	
NUMBER OF PRINTING PLATES 1	TOTAL NUMBER OF PMS COLORS 1	TOTAL AW PAGES 2	
PMS COLORS		PROCESS MATCH COLORS	
1. PMS 326 AQUA			

PACKAGE INSERT

Boston® IV
(itafocon B)
Boston® II
(itafocon A)

Spherical & Aspherical Contact Lenses for Myopia & Hyperopia, Keratoconus Contact Lenses, Bifocal Contact Lenses for Presbyopia and Toric Lenses to Correct Astigmatism in Not aphakic and Aphakic Persons

Silicone Acrylate Rigid Gas Permeable Contact Lenses For Daily Wear

IMPORTANT: Please read carefully and keep this information for future use. This package insert is intended for the eyecare practitioner, but should be made available to patients upon request. The eyecare practitioner should provide the patient with the patient instructions that pertain to the patient's prescribed lens.

CAUTION: Federal (USA) Law restricts this device to sale by or on the order of a licensed practitioner.

DESCRIPTION	Boston IV
Boston® IV (itafocon B) and Boston® II (itafocon A) are Rigid Gas Permeable Contact Lens materials composed of methyl-methacrylate-dimethylsiloxane-siloxanyl copolymer. Boston IV lens materials are available in clear and blue. The blue Boston IV lens materials contain the color additive D & C Green No. 6. Boston II lens materials are available in clear, blue and green. The blue Boston II lens material contain the color additive D & C Green No. 6; the green Boston II lens material contain the color additive D & C Green No. 6 and C.I. Solvent Yellow No. 18.	PHYSICAL PROPERTIES
	Hardness (Rockwell R) 1.17
	Specific Gravity 1.10
	Refractive Index 1.469
	Light Transmittance (400-700 nm) >90%
	Water Content <1%
	Oxygen Permeability 26* (18**)
	Wetting Angle 17*
	Boston II
	Hardness (Rockwell R) 1.19
	Specific Gravity 1.13
	Refractive Index 1.471
	Light Transmittance (400-700 nm) >90%
	Water Content <1%
	Oxygen Permeability 18* (11**)
	Wetting Angle 20*
	*gas to gas method
	**polarographic method ISO/Fatf
PHYSICAL PROPERTIES	Boston II
	Hardness (Rockwell R) 1.19
	Specific Gravity 1.13
	Refractive Index 1.471
	Light Transmittance (400-700 nm) >90%
	Water Content <1%
	Oxygen Permeability 18* (11**)
	Wetting Angle 20*
	*gas to gas method
	**polarographic method ISO/Fatf
Spherical Lens Design	
Power Range	-20.000 to +20.000 in 0.250 increments
Diameter	7.0 mm to 11.5 mm
Base Curve Range	5.00 mm to 9.00 mm in .01 mm increments
Aspherical Lens Design	
<i>(Some of these designs are patented; manufacture of these lenses in Boston IV (itafocon B) and/or Boston II (itafocon A) materials are authorized to licensed labs only)</i>	
Power Range	-20.000 to +20.000 in 0.250 increments
Diameter	7.0 mm to 11.5 mm
Base Curve Range	6.00 mm to 9.20 mm in 0.01 mm increments
Bifocal Lens Designs	
<i>(Some of these designs are patented; manufacture of these lenses in Boston IV (itafocon B) and/or Boston II (itafocon A) materials are authorized to licensed labs only)</i>	
Power Range	-20.000 to +20.000 in 0.250 increments
Diameter	6.50 mm to 9.50 mm in 0.01 mm increments
Base Curve Range	6.00 mm to 9.20 mm in 0.01 mm increments
Segment Heights	2.00 mm to +1.00 mm in 0.5 mm increments
Add Powers	+1.00D to +2.75D in 0.50 increments
Prism Ballast	0.5 to 5.5 prism diopters in .50 increments
Toric Lens Designs	
Power Range	-20.000 to +20.000 in 0.250 increments
Diameter	7.0 mm to 11.5 mm
Base Curve Range	6.80 mm to 9.50 mm in 0.01 mm increments
Toricity	Up to 9.00 Diopters
Keratoconus Lens Designs	
Power Range	-20.000 to +20.000 in 0.250 increments
Diameter	7.0 mm to 11.5 mm
Base Curve Range	4.00 mm to 9.00 mm in 0.01 mm increments
Base Optic Zone	5.00 mm to 9.00 mm in 0.01 mm increments

- Problems with contact lenses and lens care products could result in **serious injury** to the eye. It is essential that patients follow their eyecare practitioner's directions and all labeling instructions for proper use of lenses and lens care products, including the lens case. Eye problems, including corneal ulcers, can develop rapidly and lead to **loss of vision**.
- Daily wear lenses are **not indicated** for overnight wear, and patients should be instructed not to wear lenses while sleeping. Clinical studies have shown that the risk of serious adverse reactions is increased when these daily wear lenses are worn overnight.
- Studies have shown that contact lens wearers who are smokers have a higher incidence of adverse reactions than nonsmokers.
- If a patient experiences eye discomfort, excessive tearing, vision changes, or redness of the eye, the patient should be instructed to **immediately remove lenses** and promptly contact his or her eyecare practitioner.

PRECAUTIONS

Practitioner Note: Boston® IV and Boston® II contact lenses are not sterile when shipped from the Authorized Boston® Manufacturer. Prior to dispensing, clean and disinfect the lenses according to the appropriate lens care regimen.

- Special Precautions for Eyecare Practitioners:**
- Due to the small number of patients enrolled in clinical investigation of lenses, all refractive powers, design configurations, or lens parameters available in the lens material are not evaluated in significant numbers. Consequently, when selecting an appropriate lens design and parameters, the eyecare practitioner should consider all characteristics of the lens that can affect lens performance and ocular health, including oxygen permeability, wettability, central and peripheral thickness, and optic zone diameter.
 - The potential impact of these factors on the patient's ocular health should be carefully weighed against the patient's need for refractive correction; therefore, the continuing ocular health of the patient and lens performance on the eye should be carefully monitored by the prescribing eyecare practitioner.
 - Patients who wear contact lenses to correct presbyopia may not achieve the best corrected visual acuity for either far or near vision. Visual requirements vary with the individual and should be considered when selecting the most appropriate type of lens for each patient.
 - Aphakic patients should not be fitted with either Boston IV or Boston II Contact Lenses until the determination is made that the eye has healed completely.
 - Before leaving the eyecare practitioner's office, the patient should be able to properly remove lenses or should have someone else available who can remove the lenses for him or her.

- Eyecare practitioners should instruct the patient to remove the lenses immediately if the eye becomes red or irritated.
- Eyecare practitioners should carefully instruct patients about the following care regimen and safety precautions:
 - Different solutions often cannot be used together, and not all solutions are safe for use with all lenses. Use only recommended solutions.
 - Do not heat the conditioning/storage solution and/or lenses. Keep them away from extreme heat.
 - Always use **fresh unexpired** lens care solutions.
 - Always follow directions in the package inserts for the use of contact lens solutions.
 - Use only a chemical (not heat) lens care system. Use of a heat (thermal) care system can warp either the Boston IV or Boston II Contact Lenses.
 - Do not use saliva or anything other than the recommended solutions for lubricating or wetting lenses.
 - Sterile unpreserved solutions, when used, should be discarded after the time specified in the labeling directions.
 - Always keep the lenses completely immersed in the recommended storage solution when the lenses are not being worn (stored). If dry storage is desired to store the lenses for a longer period of time, they must first be cleaned, rinsed with water and carefully dried by blotting with a soft lint-free tissue prior to being placed in a clean, dry lens storage case. Ideally, these lenses should be cleaned and disinfected prior to insertion.
 - If the lens sticks (stays moving) on the eye. Follow the recommended directions on Care for a Sticking Lens. The lens should move freely on the eye for the continued health of the eye. If non-movement of the lens continues, the patient should be instructed to immediately consult his or her eyecare practitioner.
 - Always wash and rinse hands before handling lenses. Do not get cosmetics, lotions, soaps, creams, deodorants, or sprays in the eyes or on the lenses. It is best to put on lenses before putting on make-up. Water-based cosmetics are less likely to damage lenses than oil-based products.
 - Do not touch contact lenses with the fingers or hands if the hands are not free of foreign materials, as microscopic scratches on the lenses may occur, causing distorted vision and/or injury to the eye.
 - Carefully follow the handling, insertion, removal, cleaning, disinfecting, storing and wearing instructions in the Patient Instructions for Boston IV and Boston II Contact Lenses and in those prescribed by the eyecare practitioner.
 - Never wear lenses beyond the period recommended by the eyecare practitioner.
- If aerosol products such as hair spray are used while wearing lenses, exercise caution and keep eyes closed until the spray has settled.
- Always handle lenses gently and avoid dropping them on hard surfaces.
- Avoid all harmful or irritating vapors and fumes while wearing lenses.
- Ask your eyecare practitioner about wearing lenses during water activities and other sports.
- Instruct patient to inform his or her health care practitioner (doctor) that the patient wears contact lenses.
- Never use tweezers or other tools to remove lenses from the lens case unless specifically indicated for that use. Pour the lens into the hand.
- Do not touch the lens with fingernails.
- Instruct the patient to contact his or her eyecare practitioner before using any medicine in the eyes.
- Instruct the patient to inform his or her employer that he or she wears contact lenses. Some jobs may require use of eye protection equipment or may require that the patient not wear contact lenses.
- As with any contact lens, follow-up visits are necessary to assure the continuing health of the patient's eyes. The patient should be instructed as to a recommended follow-up schedule.

- ADVERSE EFFECTS**
- The patient should be informed that the following problems may occur:
- Eyes stinging, burning, itching (irritation), or other eye pain
 - Comfort is less than when lens was first placed on the eye
 - Feeling of something in the eye such as a foreign body, scratched area
 - Excessive watering (tearing) of the eyes
 - Unusual eye secretions
 - Redness of the eyes
 - Reduced sharpness of vision (poor visual acuity)
 - Blurred vision, rainbows, or halos around objects
 - Sensitivity to light (photophobia)
 - Dry eyes

If the patient notices any of the above, he or she should be instructed to:

WARNING: Boston IV and Boston II Contact Lenses are **NOT** intended for overnight (extended) wear.

- LENS CARE DIRECTIONS**
- Eyecare practitioners should review with the patient lens care directions, including both basic lens care information and specific instructions on the lens care regimen recommended for the patient.
- General Lens Care (First Clean and Rinse, Then Disinfect Lenses)**
- Basic Instructions:**
- Always wash, rinse, and dry hands before handling contact lenses.
 - Always use **fresh unexpired** lens care solutions.
 - Use the recommended system of lens care, chemical (not heat) and carefully follow instructions on solution labeling. Different solutions often cannot be used together, and not all solutions are safe for use with all lenses. **Do not alternate or mix lens care systems unless indicated on solution labeling, or if advised by the eyecare practitioner.**

- Do not use saliva or anything other than the recommended solutions for lubricating or wetting lenses. Do not put lenses in the mouth.
- Lenses should be **cleaned, rinsed, and disinfected** each time they are removed. **Cleaning and rinsing** are necessary to remove mucus and film from the lens surface. **Disinfecting** is necessary to destroy harmful germs.
- Always remove, clean, rinse, and disinfect lenses according to the schedule prescribed by the eyecare practitioner. The use of an enzyme or a cleaning solution **does not substitute for disinfecting**.

The lens care products listed below are recommended by Bausch & Lomb for use with either Boston IV or Boston II Contact Lenses. Eyecare practitioners may recommend alternate products that are appropriate for the patient's use with his or her lenses.

LENS CARE TABLE	
Product Purpose	Lens Care System
Clean	Boston ADVANCE® Cleaner or Boston® Cleaner
Disinfect	Boston ADVANCE® Comfort Formula Conditioning Solution or Boston® Conditioning Solution
Store	Boston ADVANCE® Comfort Formula Conditioning Solution or Boston® Conditioning Solution
Multi-Action Clean, Condition, Disinfect, Rinse and Cushion	Boston SMURFLIP® Multi-Action Solution
Lubricate/Rewet	Boston® Rewetting Drops
Weekly Enzymatic Cleaner	Boston® ONE STEP Liquid Enzymatic Cleaner

Note: Some solutions may have more than one function, which will be indicated on the label. Read the label on the solution bottle, and follow instructions.

- LENS CASE CLEANING AND MAINTENANCE**
- Contact lens cases can be a source of bacterial growth. Lens cases should be emptied, cleaned, rinsed with solutions recommended by the lens case manufacturer or the eyecare practitioner, and allowed to air dry. Lens cases should be replaced at regular intervals as recommended by the lens case manufacturer or the eyecare practitioner.
- CARE FOR A STICKING (NONMOVING) LENS**
- If the lens sticks (stays moving) cannot be removed, the patient should be instructed to apply one to three drops of a recommended lubricating or rewetting solution directly to the eye and wait until the lens begins to move freely on the eye before removing it. If nonmovement of the lens continues after 5 minutes, the patient should immediately consult the eyecare practitioner.
- EMERGENCIES**
- The patients should be informed that if chemicals of any kind (household products, gardening solutions, laboratory chemicals, etc.) are splashed into the eyes, the patient should: **FLUSH EYES IMMEDIATELY WITH TAP WATER, THEN REMOVE LENSES, PROMPTLY, IF POSSIBLE, AND IMMEDIATELY CONTACT THE EYECARE PRACTITIONER OR VISIT A HOSPITAL EMERGENCY ROOM WITHOUT DELAY.**
- HOW SUPPLIED**
- Each lens is supplied (monotable) in a plastic lens storage case. The case is labeled with the base curve, diopter power, diameter, center thickness, color, UV absorber (if present) and lot number. Additional parameters of add power, segment height, prism ballast and truncation may be included for bifocal lenses.

REPORTING OF ADVERSE REACTIONS

All serious adverse reactions observed in patients wearing either Boston IV or Boston II Contact Lenses or adverse experiences with the lenses should be reported to:

Consumer Affairs
Bausch & Lomb
1400 North Goodman Street
Rochester, NY 14609
1-800-333-4750

BACK SIDE